



The supraglottic airway with a non-inflating cuff



Airway management has evolved

Introducing the i-gel: a revolutionary single use supraglottic airway from Intersurgical.

i-gel and natural airway management

The i-gel is a truly unique airway device that represents the culmination of years of extensive research and development. Everything about the i-gel has been designed to work in perfect unison with the anatomy; the i-gel design was inspired by the physiology of the perilaryngeal framework itself - airway management as nature might have intended.

i-gel mirrors the anatomy

The shape, softness and contours accurately mirror the perilaryngeal anatomy to create the perfect fit. This innovative concept means that no cuff inflation is required. The i-gel works in harmony with the patient's anatomy so that compression and displacement trauma are significantly reduced or eliminated.

The non-inflating cuff


i-gel gets its name from the soft gel-like material from which it is made. It is the innovative application of this material that has enabled the development of a unique non-inflating cuff. This key feature means insertion of i-gel is easy, rapid and consistently reliable.

The simple, safe and rapid solution

i-gel is incredibly easy to use. With a non-inflating cuff, i-gel provides a safe and rapid airway management solution.

Current indications for use of i-gel

The i-gel is indicated in: Securing and maintaining a patent airway in routine and emergency anesthetics of fasted patients, during spontaneous or intermittent positive pressure ventilation, and during resuscitation of the unconscious patient, by personnel who are suitably trained and experienced in airway management techniques and devices.

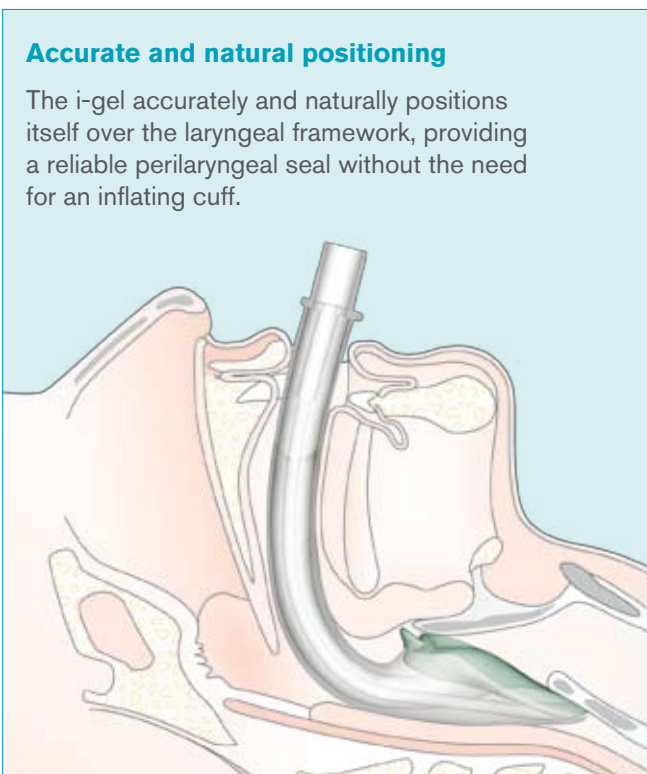

Accurate and natural positioning


The i-gel accurately and naturally positions itself over the laryngeal framework, providing a reliable perilaryngeal seal without the need for an inflating cuff.

Additional information available

An i-gel user guide, clinical study material and other support documentation is available for download from the i-gel website at www.i-gel.com

Features and benefits

i-gel has a host of features that provide significant benefits to the patient and the clinician.

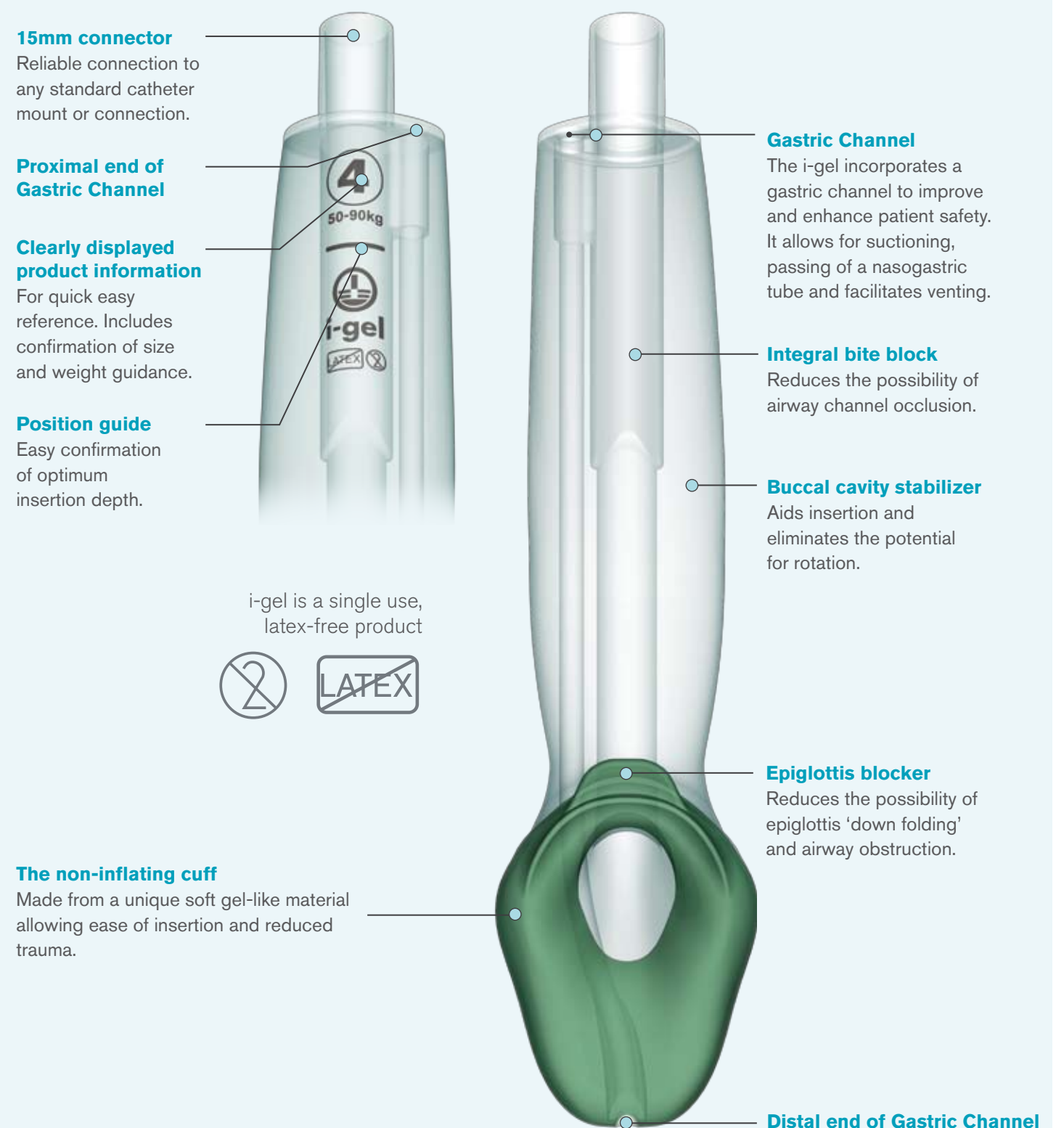

Ordering information

Innovative packaging

The i-gel supraglottic airway is supplied in a fully recyclable protective cradle. This unique packaging protects the i-gel in transit and ensures that it maintains its anatomical shape. i-gel is currently available in three sizes:

Ordering information

| | |
|---|-------------|
| Code: 8203 size 3 30-60kg (65-130 lbs.) | Box qty. 25 |
| i-gel supraglottic airway for small adults | |
| Code: 8204 size 4 50-90kg (110 - 200 lbs.) | Box qty. 25 |
| i-gel supraglottic airway for medium adults | |
| Code: 8205 size 5 90+kg (200 + lbs.) | Box qty. 25 |
| i-gel supraglottic airway for large adults | |



Find out more
www.i-gel.com

References

1. UK Resuscitation Council Advanced Life Support Guide (5th Edition). Revised June 2008.
2. Bamgbade OA, Macnab WR, Khalaf WM: Evaluation of the i-gel airway in 300 patients. *Eur J Anaesthesiol.* 2008 Oct;25(10):865-6.
3. Gatward JJ, Thomas MJC, Nolan JP, Cook TM: Effect of chest compressions on the time taken to insert airway devices in a manikin. *Br J Anaesth.* 2008 Mar;100(3):351-6
4. Wharton NM, Gibbison B, Gabbott DA, Haslam GM, Muchatuta N, Cook TM: I-gel insertion by novices in manikins and patients. *Anaesthesia.* 2008 Sep;63(9):991-5.
5. Gabbott DA, Beringer R: The i-gel supraglottic airway: A potential role for resuscitation?: *Resuscitation.* 2007 Apr;73(1):161-2.
6. Jackson KM, Cook TM: Evaluation of four airway training manikins as patient simulators for the insertion of eight types of supraglottic airway devices. *Anaesthesia.* 2007 Apr;62(4):388- 93.
7. Soar J: The i-gel supraglottic airway and resuscitation - some initial thoughts: *Resuscitation.* 2007 Jul;74(1):197.
8. Schmidbauer W, Bercker S, Volk T, Bogusch G, Mager G, Kerner T: Oesophageal seal of the novel supralaryngeal airway device i-gel in comparison with the laryngeal mask airways Classic and ProSeal using a cadaver model. *Br J Anaesth.* 2009 Jan;102(1):135-9
9. Acott CJ: Extraglottic airway devices for use in diving medicine - part 3: The i-gel: *Diving and Hyperbaric Medicine.* Volume 38
10. Liew, B. John, S. Ahmed (2008) Aspiration recognition with an i-gel airway: *Anaesthesia.* 2008 Jul;63(7):786.